

Eleview® Ease of use as described by Dr. Luigi Moro

Super: Eleview® ease of use

LM: Eleview is a property compared to other solutions. The property (factor) is the reason it's very easy to inject, because it doesn't generate resistance as it is progressing into the catheter.

Super: Dr. Luigi Moro—Chief Scientific Officer, Cosmo Pharmaceuticals

LM: You know that when you have to inject you have to pass (the solution) through a catheter that is long, 2 meters more or less, and you have a very small diameter.

This generates a resistance to the progression. Solutions that are very consistent in terms of thickness, in terms of being resistant to progression generate an incredible problem for the nurse or the endoscopist that has to push the solution inside of the catheter.

Super: Low-viscosity solution

LM: The characteristic of Eleview is to be free “flow” inside of the catheter and once in loco interacts with the ester ture or betas component in order to become viscous and then structure the cushion. This has been done in situ. The other solutions generally have the same thickness generated or so into the ampoules in vitro. So the problem is how to push—in this case—the viscous solution inside of a small catheter that has a very small diameter.

Please see Eleview® instructions for Use for complete Important Safety Information at eleviewUS.com.

