

2.

VIDEO: EMR Cecum Rex

Video Title

Dr. Douglas Rex: Endoscopic Mucosal Resection of the Cecum with Eleview®

This is a lesion in the cecum that's also serrated. And we've seen recent evidence that, even down to about 10 mms in size, EMR has an advantage in the effective resection of serrated lesions, because it provides such good delineation of the margin.

So, as the injection is proceeding, we are again using the dynamic technique of backing away.

You can see how the methylene blue is giving us a very nice delineation of the edges of the lesion.

This will help us keep track as we proceed.

Fine to re-inject, in order to get that mound in a sort of a central position with the lesion right on top of it.

In just a second, you will see, with a black line, how nicely demarcated the perimeter of the lesion is and how it's just ready for endoscopic mucosal resection.

So, Eleview® gives us a great submucosal mound, great delineation of the margins of the lesion.

Please see Eleview® instructions for Use for complete Important Safety Information at eleviewUS.com.

